Composition

Tricalcium silicate powder
Aqueous calcium chloride solution and excipients

Properties

Biodentine® is a bioactive dentine substitute from the « Active Biosilicate Technology® » innovation.

Biodentine® has mechanical properties similar to the sound dentine and can replace it both in the crown and in the root.

Biodentine® contains mainly high-purity, monomer-free mineral ingredients. Due to its mechanical and physical properties, it can be applied directly to the tooth, **without prior surface treatment of calcified tissues.**

Biodentine® creates the optimal conditions for maintenance of pulp vitality, by providing a very tight seal on the dentine surface. It guarantees **the absence of postoperative sensitivity and the longevity of restorations in vital teeth.**

Indications

When used in the crown:

- Temporary enamel restorative material.
- Permanent dentine restorative material.
- Restoration of deep and/or large coronal carious lesions (sandwich technique).
- Restoration of deep cervical and/or radicular lesions.
- Pulp capping.
- Pulpotomy.

When used in the root:

- Repair of root perforations.
- Repair of furcation perforations.
- Repair of perforating internal resorptions.
- Repair of external resorption
- Apexification.
- Root-end filling in endodontic surgery (retrograde filling).

Biodentine is not to be used for permanent obturation or sealing of the root canal.

Limits of use

- Restoration of large loss of tooth substance subjected to high stresses.
- Esthetic restoration of anterior teeth.
- Treatment of teeth with irreversible pulpitis.

Side effects

- No known side effects.

Instructions for use (for each indication)

Biodentine® mixing instructions

- Take a capsule and gently tap it on a hard surface to loosen the powder.
 - Open a capsule and place it on the stand.
 - Detach a single-dose container of liquid and gently tap on the sealed cap to force all the liquid down the container. Twist cap to open.
 - Squeeze out the entire contents of the single-dose container into the capsule. Close the capsule.
 - Place the capsule on a mixing device, such as Technomix, Tac 400 (Lineatac), Silamat, CapMix, Rotomix, Ultramat etc., at a speed of 4000 – 4200 rotations/min, and mix for 30 seconds.
 - Open the capsule and collect Biodentine® material with a suitable instrument.
- Depending on the desired application, you may handle Biodentine® with an amalgam carrier, a spatula or a Root Canal Messing Gun.
- Rapidly rinse and clean the instruments to remove any residual material.

RESTORATION:

Assess pulp vitality by the usual tests: Biodentine® is not indicated for the treatment of teeth with irreversible pulpitis

- 1) Isolate the tooth with a rubber dam.
- 2) Remove the infected dentine with a round bur and/or a hand excavator. Leave the affected dentine.
- 3) Adapt a matrix around the tooth if a wall is missing.
- 4) Prepare Biodentine® as indicated above (*Biodentine® mixing instructions*)
- 5) Insert Biodentine® in the cavity, avoiding trapped air bubbles. Condense the material and ensure good adaptation to the cavity walls and margins.
- 6) Carve the surface of the restoration.
- 7) Wait for about five minutes after insertion of the material before removing the matrix.
- 8) To optimize the mechanical properties of the material and facilitate removal of the matrix, a varnish can or even should be applied onto the surface of the restoration.
- 9) Check occlusion.
- 10) Within one week to six months after placement of Biodentine®, prepare the cavity according to the criteria recommended for the selected restorative material. The remaining Biodentine® material can be considered as sound

artificial dentine and permanently left in deep areas of the cavity and in areas adjacent to the pulp chamber. Biodentine® is compatible with all direct or indirect crown restoration techniques, and particularly with all types of bonding systems.

PULP CAPPING:

Assess pulp vitality by the usual tests: Biodentine® is not indicated for the treatment of teeth with irreversible pulpitis

- 1) Isolate the tooth with a rubber dam.
- 2) Remove the infected dentine with a round bur and/or a hand excavator. Leave the affected dentine.
- 3) Adapt a matrix around the tooth if a wall is missing.
- 4) If there is bleeding in the pulp, hemostasis must be achieved before applying Biodentine®.
- 5) Prepare Biodentine® as indicated above (*Biodentine® mixing instructions*).
- 6) Place Biodentine® directly on the exposed pulp without condensation. Ensure good adaptation to the cavity walls and margins.
- 7) Carve the surface of the restoration.
- 8) Wait for about five minutes after insertion of the material before removing the matrix.
- 9) To optimize the mechanical properties of the material and facilitate removal of the matrix, a varnish can or even should be applied onto the surface of the restoration.
- 10) Check occlusion.
- 11) Within one week to six months after placement of Biodentine®, prepare the cavity according to the criteria recommended for the selected restorative material. The remaining Biodentine® material can be considered as sound artificial dentine and permanently left in deep areas of the cavity and in areas adjacent to the pulp chamber. Biodentine® is compatible with all direct or indirect crown restoration techniques, and particularly with all types of bonding systems.

PULPOTOMY

- 1) Isolate the tooth with a rubber dam.
- 2) Remove the infected dentine with a round bur and/or a hand excavator.
- 3) Gain access to the pulp chamber and clean out the pulp.
- 4) If there is bleeding in the pulp, hemostasis must be achieved before applying Biodentine®.
- 5) Adapt a matrix around the tooth if a wall is missing.
- 6) Prepare Biodentine® as indicated above (*Biodentine® mixing instructions*).
- 7) Place Biodentine® directly in the pulp chamber and ensure good adaptation to the cavity walls and margins.
- 8) Carve the surface of the restoration.
- 9) Wait for about five minutes after insertion of the material before removing the matrix.
- 10) To optimize the mechanical properties of the material and facilitate removal of the matrix, a varnish can or even should be applied onto the surface of the restoration.
- 11) Check occlusion.
- 12) Within one week to six months after placement of Biodentine®, prepare the cavity according to the criteria recommended for the selected restorative material. The remaining Biodentine® material can be considered as sound artificial dentine and permanently left in deep areas of the cavity and in areas adjacent to the pulp chamber. Biodentine® is compatible with all direct or indirect crown restoration techniques, and particularly with all types of bonding systems.

REPAIR OF ROOT PERFORATIONS

- 1) Isolate the tooth with a rubber dam.
- 2) Prepare the root canal alternately using suitable endodontic instruments and a solution of sodium hypochlorite.
- 3) Dry the canal with paper points and use a chlorhexidine solution or a calcium hydroxide paste for disinfection between visits. Tightly seal the access cavity with a temporary cement to protect the temporary filling.
- 4) At the next visit (usually after one week), place a rubber dam and remove the temporary crown restoration. Clean the canal alternately using a solution of sodium hypochlorite and suitable endodontic instruments. Dry the canal with paper points.
- 5) Prepare Biodentine® as indicated above (*Biodentine® mixing instructions*).
- 6) Dispense Biodentine® over the perforation site using a suitable instrument.
- 7) Condense Biodentine® with a plugger.
- 8) Take an X-ray to check that the material is correctly positioned.
- 9) Remove excess material and place a temporary filling.
- 10) Complete root canal treatment at the next visit according to current recommendations.

REPAIR OF FURCATION PERFORATIONS

- 1) Isolate the tooth with a rubber dam.
- 2) Rinse the cavity with a solution of sodium hypochlorite to disinfect the area.

- 3) If there is bleeding in the pulp, hemostasis must be achieved before applying Biodentine®.
- 4) Dry the pulp chamber.
- 5) Prepare Biodentine® as indicated above (*Biodentine® mixing instructions*).
- 6) Dispense Biodentine® and condense. Perforation repair and crown restoration are performed in a single step.
- 7) Take an X-ray to check that the material is correctly positioned.
- 8) Remove excess material.
- 9) At a subsequent visit, if all clinical signs of a successful treatment are present, the possibility of a permanent restoration can be considered.

REPAIR OF PERFORATING INTERNAL RESORPTIONS

- 1) Isolate the tooth with a rubber dam.
- 2) Prepare the root canal alternately using suitable endodontic instruments and a solution of sodium hypochlorite.
- 3) Dry the canal with paper points and use a calcium hydroxide paste for disinfection between visits. Tightly seal the access cavity with a temporary cement to protect the temporary filling.
- 4) At the next visit (usually after one week), place a rubber dam and remove the temporary crown restoration. Clean the canal alternately using a solution of sodium hypochlorite and suitable endodontic instruments. Dry the canal with paper points.
- 5) Prepare Biodentine® as indicated above (*Biodentine® mixing instructions*).
- 6) Dispense Biodentine® over the resorptive defect using a suitable instrument.
- 7) Condense Biodentine® with a plugger.
- 8) Take an X-ray to check that the material is correctly positioned.
- 9) Remove excess material and place a temporary filling.
- 10) Complete root canal treatment at the next visit according to current recommendations.

APEXIFICATION

- 1) Isolate the tooth with a rubber dam.
- 2) Prepare the root canal alternately using suitable endodontic instruments and a solution of sodium hypochlorite.
- 3) Dry the canal with paper points and use a calcium hydroxide paste for disinfection between visits. Tightly seal the access cavity with a temporary cement to protect the temporary filling.
- 4) At the next visit (usually after one week), place a rubber dam and remove the temporary crown restoration. Clean the canal alternately using a solution of sodium hypochlorite and suitable endodontic instruments. Dry the canal with paper points.
- 5) Prepare Biodentine® as indicated above (*Biodentine® mixing instructions*).
- 6) Dispense Biodentine® in the root canal using a suitable instrument.
- 7) Condense Biodentine® with a plugger.
- 8) Take an X-ray to check that the material is correctly positioned.
- 9) Remove excess material and place a temporary filling.
- 10) Complete root canal treatment at the next visit according to current recommendations.

ROOT-END FILLING IN ENDODONTIC SURGERY:

- 1) Gain access to the operative site following the current recommendations in endodontic surgery.
- 2) Using a specific ultrasonic tip, prepare a root-end cavity, 3 to 5 mm deep in the apical portion of the root canal.
- 3) Isolate the area. Achieve hemostasis. Dry the cavity with paper points.
- 4) Prepare Biodentine® as indicated above (*Biodentine® mixing instructions*).
- 5) Dispense Biodentine® in the cavity using a suitable instrument. Condense Biodentine® with a small plugger.
- 6) Remove excess material and clean the surface of the root.
- 7) Take an X-ray to check that the material is correctly positioned.

Warnings and precautions for use

- Ensure that the rubber dam is properly placed so as to completely isolate the operating field.
- Water contamination slows the setting of the material. Prevent exposure to water and fluids during the initial setting stage, i.e. 10 minutes.

Storage

- Store in a dry place.

Presentations

- Box containing 15 capsules and 15 single-dose containers

For professional dental use only

Caution: Federal law restricts this device to sale by or on order of a dentist.

Biodentine®
Active Biosilicate
Technology®

Bioactive dentine substitute
Substituto dentinario bioactivo

Manufactured for/ Fabricado para:
SEPTODONT
Louisville, CO, 80027
By/ Por: SEPTODONT
94100 Saint-Maur-des-Fossés
France/ Francia

10/10

05 14 268 00 00

Biodentine®
Active Biosilicate
Technology®

Bioactive dentine substitute
Substituto dentinario bioactivo

Biodentine®
Active Biosilicate
Technology®

Bioactive dentine substitute
Substituto dentinario bioactivo

Biodentine®
Active Biosilicate
Technology®

Bioactive dentine substitute
Substituto dentinario bioactivo

Biodentine®
Active Biosilicate
Technology®

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