Cimpat[®]

Ready to use temporary filling

Properties

CIMPAT is a premixed temporary filling material conveniently supplied in syringes (CIMPAT S) or in jars (CIMPAT white and pink). It hardens readily on contact with oral fluids. There is good adherence to dentin and it expands slightly during setting to ensure minimal marginal leakage.

CIMPAT is nontoxic and not irritating to hard and soft tissues and can be safely used in close proximity to pulpal and gingival tissues.

Indications

CIMPAT can be used for the temporary restoration of cavities, and in the interval between tooth preparation and the permanent cementation of completed inlays.

Directions for use

• SYRINGES

Push out the required amount of paste by turning the screw. For ease of handling the use of a dry spatula is suggested. To prevent waste of unneeded paste, turn the screw back two or three turns to ease the pressure. Replace the stopper tightly after each use to prevent drying. Place CIMPAT gently into the cavity using minimal compression. This can be achieved with a cotton bud. Excess material from the margins should be removed with a blunt probe and the occlusion checked to ensure occlusal clearance. Patients can be dismissed while the dressing is still soft, however they should be cautioned to avoid eating, drinking and rinsing for one hour.

CIMPAT will harden within one hour in the oral cavity but the setting time may be accelerated by placing it into a warm air dried cavity.

• JARS

Use a dry spatula to take the required amount of the paste from the jar. Replace the cap tightly after each use to prevent drying.

Place CIMPAT gently into the cavity using minimal compression. This can be achieved with a cotton bud. Excess material from the margins should be removed with a blunt probe and the occlusion checked to ensure occlusal clearance. Patients can be dismissed while the dressing is still soft, however they should be cautioned to avoid eating, drinking and rinsing for one hour.

CIMPAT will harden within one hour in the oral cavity but the setting time may be accelerated by placing it into a warm air dried cavity.

Precautions for use

- Like any filling materials, CIMPAT remains in place only when the cavity is properly trimmed.
- CIMPAT is softened during setting by certain solvents contained in liquid or pastes used for canal treatment. Therefore after any canal treatment and before filling with CIMPAT, it is required to clean the cavity thoroughly and allow to dry.
- Any non-sterilizable item that is handled in the delivery of the dental service should be disinfected by standard dental office hygiene procedures.

Storage

To be stored away from moisture and heat; 50°F- 75°F/ 10°C-24°C

Caution

Federal law restricts this device to sale by or on the order of a dentist.

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