Image Caries Detector

Dye Solution For Detecting Infected Dentin

FOR PROFESSIONAL USE ONLY

Instructions for Use

INDICATIONS FOR USE:

Image Caries Detector is designed to be used to help identify caries in dentin.

DESCRIPTION:

Image Caries Detector is a dye solution. It is to be placed on the dentinal surface in question and the excess washed away.

FORMULATION:

Image Caries Detector is a food dye in propylene glycol.

INSTRUCTIONS:

Isolate the carious teeth.

Wash and dry the area in question.

Remove the cap from the bottle and apply by one of the following methods:

- a) place one drop into a mixing well and paint on the tooth with a brush;
- b) apply to a sponge pellet and spread the solution on the tooth with the pellet.

SYRINGE APPLICATION: Remove the cap from the syringe and apply the luerlocking sponged syringe tip. Gently express a small amount of material onto a mixing pad to begin the flow. Then apply directly to the tooth. Remove and discard syringe tip. Replace cap on syringe. Allow to remain on the tooth for 10 seconds. Wash thoroughly and dry. Remove the stained carious tissue with a bur in a slow speed handpiece or with a hand instrument.

Repeat the application after affected tissue is removed to help determine if any further removal is necessary. After decay is removed, restore by conventional dental restorative technique. Recap the bottle after the procedure is completed.

WARNINGS:

Do not spill or splatter the Image Caries Detector on clothing. It will stain clothing. Stains on clothing may be removed by washing the area soon after contact with detergent and warm water or with white vinegar. Do not let the Image Caries Detector contact other tissues as it will stain these tissues. Dentists and assistants should wear gloves and protective eye wear as during any normal dental procedure.

Patients also should wear eye protection. Any non-sterilizable item that is handled in the delivery of the dental service, should be disinfected by standard dental office hygiene procedures.

STORAGE CONDITIONS:

The system is designed to be stored between 50°-75°F (10°-24°C). Do not store the materials in intense light, or under wet conditions.

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