

# Endosolv E

## Root canal filling solvent

---

### Indications

To aid removal of root canal filling material from a root canal, to permit corrective treatment.

### Directions

Gain access to root canal by opening via occlusal area. Place a drop of ENDOSOLV E in the opened canal, in contact with old filling material to be removed. Using a reamer or other appropriate endodontic instrument, remove the soften filling material. Add more ENDOSOLV E repeatedly as the removal process continues, as necessary to soften old restorative material. It is frequently desirable to open the root in question, place ENDOSOLV E in the pulp chamber via a cotton pledget, reclose the tooth, and allow the ENDOSOLV E to remain in the canal for one or more days. This procedure is recommended if conditiond permit.

### Caution

When ENDOSOLV E has been used, it is necessary to thoroughly clean and dry the canal before placement of an endodontic filling.

### Presentation

Supplied as a 0.44 Oz bottle.

---

Distributor: SEPTODONT, Inc.  
245 Quigley Boulevard - Suite C  
NEW CASTLE, Delaware 19720 U.S.A.  
Made in France.

