

Pharmaéthyl[®]

Local pulp and vitality testing agent

Ingredients

Tetrafluoroethane	95.55 g
Dimethyl ether	3.40 g
Natural mint flavor	0.05 g

Indications

Topical pulp and tooth vitality testing agent.

Directions for use

Remove white cap from top of bottle.

Press enclosed nozzle firmly into tube top of package. Spray by pressing down on nozzle, directing spray as required.

Spray from a distance of 1-2 inches.

White cap can be discarded upon first use of package.

Caution

Use sparingly.

Excessive use can cause soft tissue burns.

NOT FLAMMABLE

CAUTION-CONTENTS UNDER PRESSURE

Do not puncture or incinerate container. Do not expose to heat or store at temperatures above 60°C (122°F). Do not spray close to a flame or any incandescent sources. Keep out of the reach of children.

Caution : Federal law restricts this device to sale by or on the order of a dentist.

Presentation

1 aerosol container 7.5 oz (210 g)

1 swivel type sterilizable nozzle.

Revised 07/2001

Distributor: SEPTODONT, Inc.
245 Quigley Boulevard - Suite C
NEW CASTLE, Delaware 19720 U.S.A.

Made in France by :
Septodont

