

Alvogyl®

Alveolar Dressing

Formula

| | |
|--|----------|
| Butamben | 25,70 g |
| Iodoform | 15,80 g |
| Eugenol | 13,70 g |
| Penghawar, olive oil, sodium lauryl sulfate, spearmint oil | |
| purified water, calcium carbonate ad | 100,00 g |

Description

Introduced into the socket, ALVOGYL easily adheres to the alveolus due to its fibrous consistency. Assisted by the patient's tongue movements, it gradually self eliminates without any intervention by the practitioner or the patient as the healing of the wound takes place.

Indications

ALVOGYL may be used as a post-extraction dressing following a difficult or traumatic extraction.

Contraindications

Routine use of the product as a post-extraction dressing is contraindicated.

ALVOGYL is not to be used on patients having a history of allergy reactions to procaine type anesthetics or sensitivity to iodine compounds.

Directions for use

Syringe the socket with warm normal saline solution. Then, using a pair of sterilized tweezers, apply a sufficient quantity of ALVOGYL (about 0,20 g) to adequately cover the bottom of the socket and pack gently into place.

Do not suture.

Precautions for use

Always replace cap immediately after use and keep jar in a dry place below 25° C.

Presentation

Jar containing 11 g of paste.

Storage conditions

The product should be stored below 25°C.

REG. U.S. PAT. & TM. OFF.

Distributor: SEPTODONT, Inc.
245 Quigley Boulevard - Suite C
NEW CASTLE, Delaware 19720 U.S.A.

Made in France by :
Septodont

