

Racestyptine solution

Topical hemostatic agent and aid for gingival retraction

Solution of Aluminum chloride (25%), 8-Hydroxy quinoline sulfate (0.1%) and inert excipient.

RACESTYPTINE solution is an astringent/hemostatic for use topically in the oral cavity. It is especially beneficial:

As a gingival retraction aid.

For temporary control of light gingival bleeding.

As a pre-impression application on the gingival margins, associated with the taking of indirect impressions for inlays, onlays, etc.

As a gingival retraction aid, RACESTYPTINE solution is recommended for use directly on the gingiva by itself, or in conjunction with plain retraction cords, with RACESTYPTINE RINGS, or as a supplemental application with RACESTYPTINE retraction cords. The exact application depends on the specific case and on the operator's choice of technique.

Directions for use

Apply RACESTYPTINE solution topically as needed, via retraction cord, a saturated cotton pledget, eye dropper, or other suitable applicator.

Keep package closed when not in use.

Caution : U.S. Federal law prohibits dispensing without prescription. For use by, or under the direction of, a dentist. Not for ingestion or injection.

Supplied as a 0.44 fl oz (13 mL) bottle.

Distributor : SEPTODONT, Inc.
245 Crisley Boulevard - Suite C
NEW CASTLE, Delaware 19720 U.S.A.

Made in France by : Septodont

