0104020014 - SeptoWedges - 100 Einheiten nr. 4 - violett (15 mm)
0104020019 - SeptoWedges - 100 Einheiten nr. 5 - natürlich (16 mm)

SeptoWedges

Interdental wooden wedges

SeptoWedges are indicated for general use in restorative procedures, with the mechanical function of adapting matrices and/or separating adjacent teeth. The SeptoWedges do not require cuts or adjustments, because its shape allows an easy insertion into the interdental areas and ensures a perfect adaptation of the matrices.

Use: Select the most appropriate wedge size for the procedure. Insert the wedge into the interdental space. Once the procedure is completed, the wedge should be removed.

Warnings, precautions and restrictions: Professional use only. Single use only. Its reuse may cause cross contamination. If there is proven allergy to any of its components, the product must not be used. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operative field. The product has a sharp point and must be handled with caution to avoid risk of causing harm to the patient.
Composition: Natural: Wood. Colored: Wood and non-toxic reactive dyes.
Storage: To be stored in dry and fresh place, protected from sunlight.

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0104020014 - SeptoWedges - 100 assorted colored units (5 sizes)
0104020015 - SeptoWedges - 100 pieces nr1 - yellow (12 mm)
0104020016 - SeptoWedges - 100 pieces nr 2 - green (13 mm)
0104020017 - SeptoWedges - 100 pieces nr 3 - red (14 mm)
0104020018 - SeptoWedges - 100 pieces nr 4 - violet (15 mm)
0104020019 - SeptoWedges - 100 pieces nr 5 - naturel (16 mm)

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