**SEPTOWHEELS PX**

Impregnated felt disc for final gloss

SeptoWheels PX are composed by felt impregnated with dehydrated abrasive paste, extra-line aluminum oxide based, indicated for polishing of composites and dental enamel. The abrasive paste is activated with a few drops of water, providing excellent polishing and natural sheen. SeptoPlus Mandrel system that works by pressure, easy and fast to set, provides agility in the use.

**Use:**
Instal the mandrel onto the contra-angle. Connect the support of the felt disc in the mandrel's end. Humidity the SeptoWheels PX disc with a few drops of water to activate the abrasive paste. Apply the product in low rotation, exerting low pressure and with circular and intermittent touches.

**Warnings, precautions and restrictions:**
Professional use. Single use only. Its reuse may cause cross contamination. SeptoWheels PX should be used in rotations until 4.000 rpm. The use of the mandrel with similar products is of the user's sole responsibility. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. If there is proven allergy to any of its components, the product must not be used.

**Cleaning and Sterilization:**
The mandrel is autoclavable (126°C, 147kPa, 16min.). Before sterilization, proceed with the cleaning using water and enzymatic detergent. Themandrel durability is directly related with the care during its use.

**Composition:**
Discs: Aluminum oxide and wool felt. Fixing System: Polyvinyl chloride. Mandrel: Metal.

**Storage:**
To be stored in dry and fresh place, protected from sunlight.

**Connection with contra-angle:**
The SeptoPlus Mandrel is manufactured in compliance with specifications of the rule ISO 1797.

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