

ORA[®] SOOTHE[®]

"SOCKIT" GEL



PATIENT INSTRUCTION SHEET

- Remove resealable cap, apply the gel to the affected area, and reseal the tip by snapping into place.



- Start using OraSoothe SockIt Gel the day of procedure. Apply as often as needed for pain relief, but at least 4 times each day until the syringe is empty.
- Do not rinse with salt solution or any other product, as this will rinse the gel out.
- OraSoothe SockIt Gel contains no drugs, and is non-toxic and safe when swallowed.
- Store OraSoothe SockIt Gel at room temperature (between 50°- 86° F); avoid excessive heat or freezing. Do not store in direct sunlight. Change in color is expected.
- If you have any questions regarding the use of SockIt Gel, please call 800.872.8305 or visit www.orasoothe.com.

Manufactured by: MCMP
230 Westway Pl. Ste 111
Arlington, TX 76018
p: 800.755.0044 f: 888.690.8744

Distributed in USA by:
SEPTODONT Inc
205 Granite Run Dr., Suite 150
Lancaster, PA 17601, USA

Distributed in CANADA by:
SEPTODONT
25 Wolseley Ct, Cambridge
ON N1R 6X3, Canada

www.orasoothe.com

SI-PAT-RV16